

Cryo EMS Machine

Cryo Pad Body Sculpting Machine

Model No. TB-Cryo10A-1



Brief Description

T&B Beauty Cryo EMS Machine integrates cutting-edge Cryolipolysis (fat freezing), RF Thermal Tightening, and Electrical Muscle Stimulation (EMS) into one powerful platform. It delivers unparalleled versatility and high performance for professional aesthetics clinics and medical spas.

Key Features

- Cryolipolysis and EMS can work simultaneously or independently.
- The parameters of the 4 pads can be adjusted independently.
- -10 °C for freezing fat, 45 °C for skin tightening, EMS for muscle sculpting.
- Three technologies in one platform — Cryo + RF + EMS.
- 8-inch touch colour screen with preset and custom programs.

Technical Specifications

Technology	Cryo + EMS + Heating (RF)
Screen	8-inch Touch Colour Screen
Cryo Pads	4 pcs (independently controlled)
Cooling Temperature	-10 °C to 5 °C
Heating Temperature	35 °C to 45 °C
EMS Frequency	1 – 100 Hz
Function 1	Fat burning / Body sculpting
Function 2	Muscle relax / Skin tightening
Applications	Abdomen, flanks, thighs, arms, buttocks, back

Detailed Description

The Ultimate Triple-Action Cryo & EMS Body Sculpting Solution

Say goodbye to single-modality limitations and embrace the revolution in efficient, intelligent and customizable body contouring. The T&B Beauty Cryo EMS Machine integrates cutting-edge Cryolipolysis (fat freezing), RF Thermal Tightening and Electrical Muscle Stimulation (EMS) into a single powerful platform, delivering unmatched versatility for professional aesthetics clinics and medical spas.



Advantages: Triple Action, Multiplied Efficiency

Conquer Fat & Muscle Simultaneously

TB-Cryo10A-1 exclusive technology supports the simultaneous operation of Cryolipolysis (Cryo) and Electrical Muscle Stimulation (EMS). A single session effectively disrupts fat cells, tightens skin contours and sculpts muscle lines — treatment time is reduced by up to 30% while client satisfaction rises accordingly.

Precise Zoning & Personalized Customization

The 4 independent Cryo Pads feature individual parameter control systems. Temperature, EMS intensity and mode can be tailored independently for each pad based on the fat thickness, skin condition and muscle group of different areas (stubborn abdomen, inner thighs, upper arms, etc.), enabling truly personalized contouring plans.

Ice-Hot Dual-Action, Amplified Results

-10 °C deep freezing targets and crystallizes fat cells, triggering their natural apoptosis. 45 °C intelligent RF heating stimulates collagen fibres to instantly contract and reorganize, tightening the skin from within. The alternating cold-hot action improves circulation and accelerates fat metabolism for visible, longer-lasting results.

Smart Operation, Effortless Profit

The intuitive 8-inch touchscreen interface provides both preset and custom programs at the touch of a button. Operators of any experience level can deliver consistent, professional treatments while maximising throughput and profitability.

An advertisement for the ADVANTAGE device. On the right, a woman in a white sports bra and shorts is shown from the waist up, with her hand on her hip. On the left, the ADVANTAGE device is shown, a white rectangular unit with a touchscreen interface and various ports. The word 'ADVANTAGE' is prominently displayed at the top left. Below it, a list of features is provided, each preceded by a checkmark.

ADVANTAGE


- ✓ 4 cooling pads with 8 refrigerants, the lowest temperature can reach -10 degree, continue to work without downtime.
- ✓ The EMS function accelerates fat metabolism.
- ✓ The cooling pads can work simultaneously or separately.
- ✓ The output of microelectricity medical modes is in line with the principle of human bionics which is safe and effective.
- ✓ Using semiconductor water circulation cooling system, the heat dissipation system is stable. 3.5L water tank.

Core Technology: Scientific Sculpting, Intelligent Control

Triple-Synergy Technology


- Industry breakthrough: overcomes traditional limitations where Cryo and EMS or heating cannot run together.

- Synergistic enhancement: Cryo accelerates fat breakdown and metabolism, RF heat instantly tightens skin, and EMS deep-activates muscles.
- King of efficiency: fat reduction, skin tightening and muscle sculpting achieved in a single session.

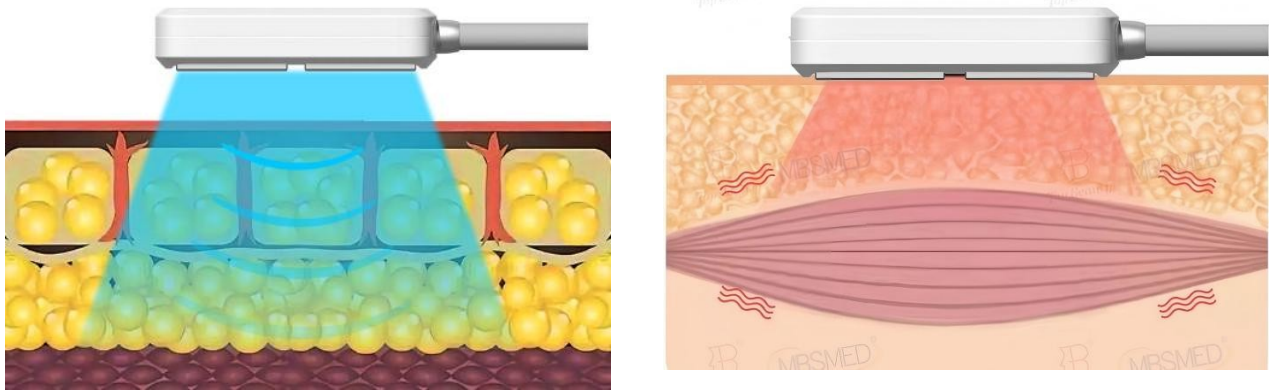

Smart 4-Channel Independent Control System

- Precision control: each of the 4 high-performance Cryo Pads features its own microprocessor.
- Temperature range: -10 °C to 45 °C — strong freezing for thicker fat, gentler heating for delicate areas.
- EMS mode & intensity: multi-level adjustable stimulation programs for every muscle group.
- Simultaneous or independent Cryo / RF / EMS operation per pad.
- Ideal for treating multiple body zones in one appointment.

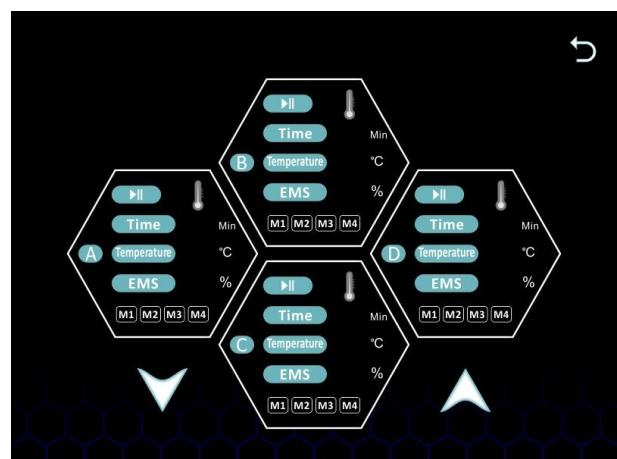

Advanced Cryolipolysis (-10 °C Fat Freezing)

- Targeted fat crystallization: precisely controlled -10 °C temperature selectively targets subcutaneous fat cells, causing their apoptosis.
- Patented temperature control & safety monitoring: real-time sensors together with contact detection safeguard the skin.
- Efficient metabolism: apoptotic fat cells are naturally metabolized and cleared by the lymphatic system.



RF Thermal Tightening (45 °C Skin Tightening)

- Instant collagen shrinkage & renewal: precise 45 °C RF heat penetrates the dermis, causing collagen fibres to contract immediately and stimulating long-term regeneration.
- Comfortable experience: intelligent temperature control ensures even heat distribution — clients experience a warm, relaxing sensation.

EMS Muscle Sculpting

- Deep muscle activation: safe, controlled electrical impulses penetrate the skin and intensely stimulate target muscles.

- Muscle toning & definition: enhances muscle contraction, lifts buttocks and defines lines.
- Boosts metabolism & fat burning: intense muscle activity consumes glycogen and fat, further accelerating results.
- Multi-level, multi-mode adjustable intensity for personalized programs.
- Comfortable and non-invasive — no downtime required.



Why Choose T&B Beauty Cryo EMS Machine?

- Exceptional efficacy: Triple-Synergy technology delivers combined fat reduction, skin tightening and muscle sculpting in one session.
- Superior efficiency: independent 4-channel design plus simultaneous mode significantly boosts treatment throughput.
- Ultimate flexibility: independent parameter control meets all needs, from delicate muscle sculpting to strong fat reduction.
- Client comfort & safety: precise temperature control, real-time monitoring and ergonomic pad design ensure a comfortable treatment.

- Robust build & reliable performance: medical-grade components engineered for daily clinical use.

EMS Cryo Plate Features

Intensively reduce fat, raise buttocks healthily, and shape a charming and good figure



Before and After Results



Product Gallery







Voltage AC110v/220v 50-60HZ

Power 400W

Fuse F3AL250V

Screen 8 inch touch color screen

cooling temperature -10°C ~ 4°C

EMS pulse width 200us


EMS frequency 2 ~ 100 Hz

Cooling handle 4 slices

Working humidity ≤80%

Working temperature 5°C~40°C

Parts Introduction

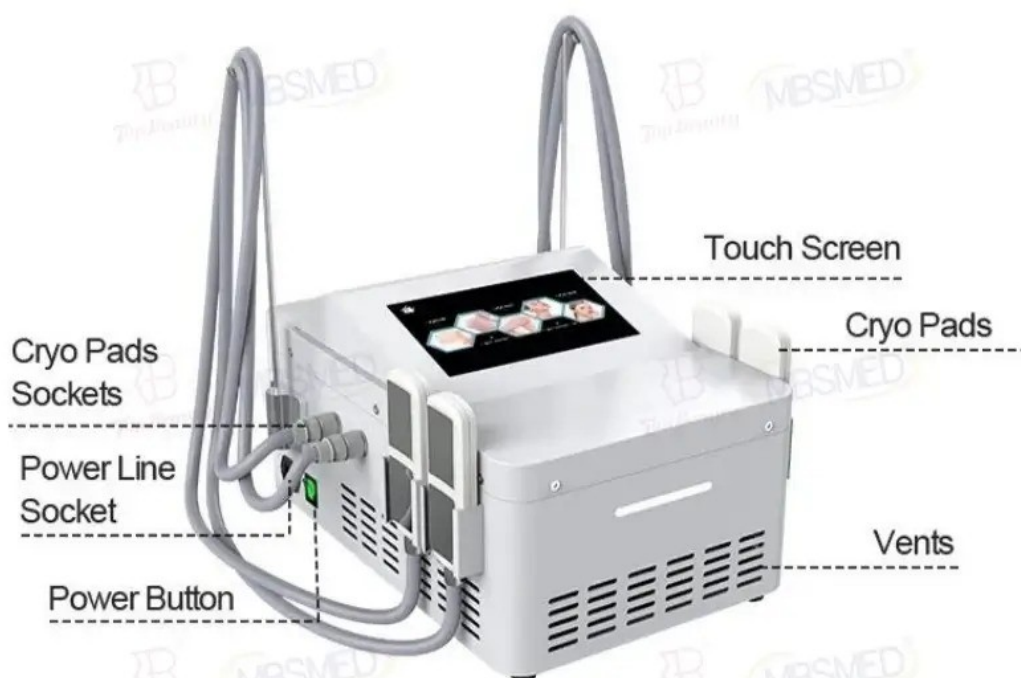

Contact

Website: <https://www.mbs-med.com/>